BébéSounds®

Prenatal Gift Set

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Model #BE005

Instruction Manual
Manual de Instrucciones

TABLE OF CONTENTS

Please Read the Following to Enjoy Our Product.....4

SECTION I: LISTENING TO YOUR UNBORN BABY

Introduction.....5

Contents.....6

Identifying the Fetal Sounds You Will Hear.....7

When You Will Hear These Sounds.....7

Prenatal Heart Listener® vs. Doppler.....8

Record Your Baby's Sounds and Your Own Heartbeat.....8

Instructions for Listening to Your Unborn Baby.....9

Instructions to Record Your Baby's Sounds.....11

Instructions to Record and Email
Your Baby's Sounds with Your Computer.....11

SECTION II: TALKING & PLAYING MUSIC TO YOUR UNBORN BABY

Introduction.....12

How Your Unborn Baby Develops.....13

How Your Unborn Baby's Brain Develops.....14

Research Findings.....	15
Instructions for Playing Music to Your Unborn Baby.....	17
Playing Music to Your Newborn Baby.....	19
Instructions for Talking, Singing, and Reading to Your Unborn Baby.....	19
Frequently Asked Questions.....	20
Endnotes.....	21
Schedule of Music Played.....	22

**SECTION III:
ADDITIONAL PRODUCT INFORMATION**

Tips for Using the Prenatal Heart Listener®.....	23
Care and Cleaning.....	23
Troubleshooting.....	24
Technical Specifications.....	26
Disclaimer.....	27
Limited Product Warranty.....	27
Also Available from Bébésounds®.....	30
En Español.....	32

PLEASE READ THE FOLLOWING TO ENJOY OUR PRODUCT

- Please have patience. This product requires a little time but the results are wonderful. You should start using it at the beginning of your third trimester.
- What you will hear through our Prenatal Heart Listener® will be the natural sounds of your baby. Your doctor uses Doppler ultrasound equipment which produces artificial noise by turning movement into sound. Our Prenatal Listener® amplifies the actual sounds of your baby.
- The best time to listen is 3 to 4 hours after eating and when you are relaxed.
- The Prenatal Heart Listener® must be used in a very quiet room.
- Placing the volume control at a low to moderate level will produce the best results and avoid static.
- You might not hear your baby's sounds immediately so don't be disappointed. The more you use the Prenatal Heart Listener®, the better you will become at finding the sounds. Your baby's position will also greatly affect your success. The best position is with your baby's back to your belly button. Your success will also depend on how relaxed you are. You may have to try several times before you will hear your baby's sounds, particularly the heartbeat.
- Your baby's heartbeat does not sound like your own. A fetal heart beats from 120-180 times per minute. It will sound like a soft, rapidly beating drum or a galloping horse.
- You will also hear kick, hiccups, and nutrients passing through the placenta. Kicks sound like uneven thumps, hiccups like two rapid drumbeats, and the nutrients like a whooshing sound.

- The provided headset is specially designed to be used with your Prenatal Heart Listener®. Do not use any other headset.
- Our Prenatal Heart Listener® is completely safe for you and your baby. It does not use any AC current, ultrasound, or radio waves.

SECTION I: LISTENING TO YOUR UNBORN BABY

INTRODUCTION

Thank you for purchasing our BébéSounds® Prenatal Heart Listener®. Please read the following material carefully to maximize your enjoyment of this product.

It is important to understand that many factors affect what you will hear with your Prenatal Heart Listener®. The position of your baby, your baby's weight, your weight, your stage of pregnancy, and where you place the Prenatal Listener® will all affect what you will hear.

For example, when listening for the fetal heartbeat, the sound will be loudest in the third trimester and if your baby is positioned with her back to your belly button. Your baby may naturally shift position, thereby creating the situation where you will hear your baby one moment but not the next. However, once you've passed the second trimester, your baby will be growing very rapidly and you will hear the heartbeat and all the other sounds on a much more regular basis.

Don't worry! It's quite normal that there will be times you will not be able to hear your baby's sounds due to your baby's position or stage of development. This is especially true of the heartbeat which is highly dependent on your baby's position.

CONTENTS

fig.1

prenatal heart listener*

fig.2

foam cover

fig.3

two headsets

fig.4

recording cable

fig.5

fetal speakers

fig.6

fetal microphone

fig.7

0

fig.8

prenatal & postnatal CD

fig.9

adapter

IDENTIFYING THE FETAL SOUNDS YOU WILL HEAR

You will hear kicks, hiccups, nutrients passing through the placenta, and, of course, the heartbeat. Each one is distinctive and being able to identify them will significantly enhance your experience.

The best way to describe each sound is as follows:

- Heartbeat: A very soft rapidly beating drum or a galloping horse
- Kicks: A series of uneven thumps
- Hiccups: Two rapid, soft drumbeats
- Nutrients: A whooshing sound

A fetal heart beats from 120 to 180 times a minute while the heart of a pregnant woman beats approximately 100 times a minute. Knowing this should help prevent you from confusing your own heartbeat with that of your baby's.

WHEN YOU WILL HEAR THESE SOUNDS

The fetal heartbeat, kicks and hiccups may be heard from the beginning of your third trimester. You will need to listen very carefully for your baby's heartbeat as it may be extremely difficult to hear it over the beating of your own heart. The sound of your own bowels may also interfere with your ability to hear the heartbeat. While the Prenatal Heart Listener[®] is specially designed to tune out these sounds, they can still get in the way. This is why the best time to listen is 3 to 4 hours after a meal and when you are relaxed and have a slower heartbeat.

Remember, as your baby grows and gets stronger, so do the sounds she creates. You will also become more familiar with the sounds so they will be easier to identify.

PRENATAL HEART LISTENER® VS. DOPPLER

You will notice that the sounds you hear through the Prenatal Heart Listener® are different from those you hear at your doctor's office. This is because your doctor probably uses Doppler ultrasound equipment which creates electronic noise based on movement. The Prenatal Listener®, however, is designed to amplify the sounds naturally by using the absolutely safe Bell stethoscope method of amplification. Again, the heartbeat heard through the Prenatal Listener® sounds more like the soft, rapid beating of a drum, while the Doppler sound is more of a gushing noise. Please keep in mind that it will take a little practice to pick up this faint sound so listen very carefully.

RECORD YOUR BABY'S SOUNDS & YOUR OWN HEARTBEAT

With our recording cable and your own recorder you can record your baby's sounds. We also encourage you to record your own heartbeat prior to your baby being born. You will then be able to play it back to your newborn after birth which may have a calming and soothing effect.


INSTRUCTIONS FOR LISTENING TO YOUR UNBORN BABY

(See Figs. 10 and 11)

1. Remove the battery cover and install a new 9-volt battery (not included). We recommend using a new battery as an old one will severely interfere with the performance of your Prenatal Heart Listener[®]. Be sure to match positive (+) to positive (+) and negative (-) to negative (-). Replace the battery cover.
2. Insert the headset plug into either jack on the unit and put the headset on. For two people to listen at the same time, insert the second headset plug into the other jack.
3. Be sure the foam cover is in place on the listening cone. Hold the listening cone to your stomach or lower back. Use the Prenatal Listener[®] on bare skin in a quiet room.
4. Turn the Prenatal Heart Listener[®] on by turning the Power/Volume control wheel to the right until you hear a click. The red power indicator on the front will light up to indicate the unit is in the ON position.
5. Adjust the volume to optimize the listening level and amplification. Begin with a low to moderate level to avoid static.
6. Start by placing the listening cone two inches below and several inches to the right of your belly button. Gently move the unit one inch at a time to different places until you find the best position for listening to your baby's sounds. Before moving the Prenatal Listener[®], turn the volume level down to the lowest position to avoid the sound of the cone moving against your skin.
7. When you are finished, turn the Power/Volume control wheel all the way to the left until you hear a click to turn the Prenatal Listener[®] off. The red power indicator light will turn off.

fig.10

power/volume control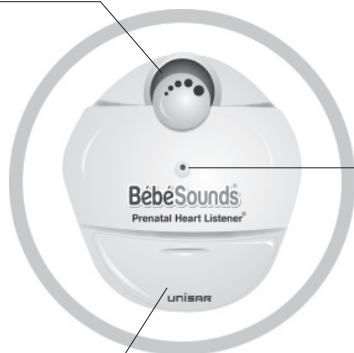


power indicator

battery compartment

fig.11

power/volume control

listening cone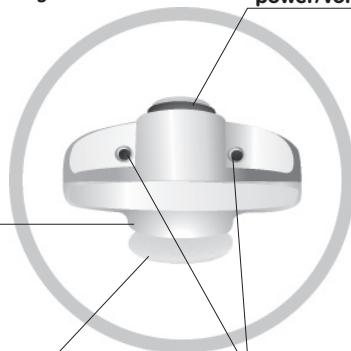

foam cover

headset jack

INSTRUCTIONS TO RECORD YOUR BABY'S SOUNDS

(See Figs. 10 and 11)

1. Plug the L-shaped end of the recording cable into either jack of the Prenatal Listener[®]. Plug the straight end into the line input of your recorder (some portable recorders do not provide this input).
2. Set up your Prenatal Heart Listener[®] for listening to your baby.
3. Adjust the recording level on your recorder if necessary.
4. Hold the Prenatal Listener[®] very still and begin recording. Remember, the sound of your heartbeat can have a very calming and soothing effect on your baby. You may want to record your own heartbeat before your baby is born. You will then be able to play back your recorded heartbeat to your baby after birth. We recommend a 30-minute recording. **NOTE:** You will need to reduce the volume when you record your own heartbeat since it is much louder than that of your unborn baby's.

INSTRUCTIONS TO RECORD & EMAIL YOUR BABY'S SOUNDS WITH YOUR COMPUTER

(See Figs. 10 and 11)

1. Plug the L-shaped end of the recording cable into either jack on the Prenatal Listener[®]. Plug the straight end into the microphone jack on your computer. This is indicated by a picture of a microphone near the jack or the jack is color coded pink on the back of your computer.
2. On your computer, go to **START, PROGRAMS, ACCESSORIES, ENTERTAINMENT (or MULTIMEDIA), SOUND RECORDER.**

3. Set up your Prenatal Heart Listener® for listening to your baby.
4. Press the red recording button in the SOUND RECORDER program to begin recording a one-minute sound file. You are now recording your baby's sounds.
5. After one minute, the recording will finish. Name and save your sound file in an appropriate folder. You can now email your sound file to family and friends.

SECTION II: TALKING & PLAYING MUSIC TO YOUR UNBORN BABY

INTRODUCTION

Research into the benefits of stimulating your unborn baby has been done worldwide and it is now believed that your baby can learn while still in your womb. While most of the attention in the past has focused on the positive results of infant stimulation, it is very important not to overlook the potential benefits of prenatal stimulation.

Studies have shown that a prenatally stimulated baby tends to display faster visual, linguistic, and motor development than a baby who was not stimulated. Research has also shown that the brain of the prenatally stimulated baby tends to be more developed, thereby providing a better foundation for the child to have a higher IQ and intelligence level.

Prenatal stimulation can come from playing classical music, talking, or singing to your baby, and even rubbing your stomach. "The most effective way to stimulate your baby and enhance her brain development while still in the womb is through sound."¹

Dr. Norman A. Ravski, Assistant Clinical Professor, Department of Obstetrics and Gynecology at Yale University School of Medicine, and Attending Physician at Yale New Haven Hospital, has endorsed playing classical music to your baby. In his endorsement, Dr. Ravski states, "In-utero fetal acoustic stimulation has been shown to improve pregnancy outcomes as well as benefit the growing fetus." The Bébésounds® Prenatal Gift Set enables you to direct our specially selected classical music through your womb to your baby with our Fetal Speakers.

It is also believed that your newborn baby will recognize the voices of her parents immediately after birth if they are heard while in the womb. In addition to stimulating your baby, talking to her will help your baby's father bond with her. Talk and sing to your unborn baby using the Bébésounds® Fetal Speakers and Microphone, which will amplify your voices so they will be heard over the other noises inside the womb.

When speaking to your baby, keep in mind that the first step in your baby's development of language skills is the recognition of intonations so it is important for both parents to speak to their child. Although the mother's voice tends to be more melodic and is constantly heard, the father's voice is louder and deeper and penetrates the womb more easily.

It is always a good idea to keep your doctor informed of what you are doing during your pregnancy. Therefore, we encourage you to inform your doctor that you will be stimulating your unborn baby with the Bébésounds® Prenatal Gift Set.

HOW YOUR UNBORN BABY DEVELOPS

First Trimester: The embryo begins to develop; the head and limbs start to form; head, body, and limbs begin to move (you won't feel this movement yet); nervous system that deals with equilibrium and spatial

relations begins to develop; and heart begins to beat more strongly.

Second Trimester: The head develops more actively than the rest of the body; eyes respond to light; key organs mature; fetus reacts to music and loud noise; arms and legs are moving strongly enough so you can feel them; fetus can hear and the ability to learn significantly increases; signs of memory and the ability to respond to various stimuli are apparent; and music has a calming effect while harsh sounds have an agitating effect.

Third Trimester: The nervous system has matured and becomes fully formed; brain enters a rapid growth phase; movements are more coordinated and muscles are stronger; fetus moves to the rhythm of music; sight has developed and response to light is evident; movement is less frequent but stronger toward the end of the ninth month; and growth and development continue until birth.

HOW YOUR UNBORN BABY'S BRAIN DEVELOPS

Your baby's brain begins developing almost immediately and by the fifth month the number of brain cells your baby will have for the rest of her life is determined. From this point forward, her brain develops by increasing the size and complexity of the cells already created. Around the eighth month, your unborn baby's brain changes significantly. At this time, it doubles in weight and the connections between the brain cells, known as synapses, become even more complex.

The following is the structure of a brain cell:

Dendrites: receive messages from the body

Cell body: decides whether to send the message received from the dendrites or to store it

Axons: transmit messages received from the cell body

As the cells get larger, the axons grow longer and the dendrites become an increasingly complex system of multiple connections like branches of a tree. The more stimulation the cells receive, the more complex these dendrites become, resulting in a more developed brain.

The number of brain cells your baby is born with helps determine her potential, but it is also the number of connections between these cells that determines her final intelligence. Helping to increase the number of connections and complexity of the brain cells with stimulation increases the probability of your baby having a higher intelligence level and IQ.

RESEARCH FINDINGS

It is now known that stimulation of the senses affects the growth of the brain cells and if a fetus has little stimulation in the womb, fewer connections are made. Specialists in fetal and infant brain development have agreed that stimulating your unborn baby's senses can affect the development of the connections between the brain cells. In fact, the lack of these connections can actually cause cells to die, especially in the eighth month of pregnancy.²

By the end of the fifth month, your baby can hear and begins to learn, and you can tell when she is reacting to external stimulation. In one case, pregnant mothers actually had to leave a concert of classical music before it was over because their babies were kicking so hard in time to the music that they couldn't tolerate it.³

In another case, mothers were asked to play their favorite classical music to their unborn baby for ten minutes a day at a volume that was just a little louder than normal speaking voice. It was discovered that the babies recognized the change in their environments immediately as was evidenced by their heart rates increasing noticeably. What is also

interesting is that the unborn baby does not respond in this way to many single notes even if they are played very loud and very close to her.⁴

Yet another study showed that unborn babies who had classical music played to them from the middle of the fifth month until they were born, for ten minutes twice a day, developed more quickly, began to talk up to six months earlier, and had greater intellectual development than those babies who had not had classical music played to them inside the womb.⁵

Some women worry that if they improve the brain's development, the baby's head will be larger thereby causing an increase in the risk of delivery problems. In fact, there is reason to believe the opposite is true. Prenatal sound stimulation has led to shorter and less painful labor periods, a lower number of cesarean section deliveries, and a greater number of non-traumatic births.⁶ In addition, a baby who experiences prenatal stimulation tends to:

- Be calmer and more alert
- Be happier and cry less
- Have a longer attention span
- Be stronger and have better coordination
- Lift his or her head up earlier and stand earlier
- Walk earlier
- Talk earlier
- Have more self-confidence⁷

Remember, by the end of the fifth month your unborn baby begins to hear and by the sixth month, her ability to learn increases significantly. It is now time to start stimulating your baby. Keep in mind that your baby is most alert during the evening hours so you may want to play music or talk to her a few more times.

INSTRUCTIONS FOR PLAYING MUSIC TO YOUR UNBORN BABY (YOU WILL NEED YOUR OWN CD PLAYER)

The best time to play music is once in the morning and once in the evening, for a period of 5 to 10 minutes, around the same time each day. Be careful not to play the music too long or too frequently as your unborn baby also needs to rest. However, you will probably find that your baby will be very active and obviously awake during the evening.

Do not play the same music over and over again. It is believed that if your baby hears the same thing repeatedly, she will eventually ignore it. However, periodic repetition of the same music is advisable. In some cases, when the music was played after the baby was born, the newborn seemed to recognize the melody.

We suggest playing different music during each session in a single day and replaying it two or three days later. We also suggest that you keep a schedule of the music you have played showing the date, music played, and times it was played. We've provided you with a starter schedule on page 22 in this booklet.

To play music to your unborn baby, follow these easy steps (See Fig. 12):


1. Connect the cable from the Fetal Speakers to the input jack on your own CD player. If you want to listen to the music at the same time, use the enclosed adapter. Plug the adapter into your CD player. Then plug the cable from the Fetal Speakers into one of the jacks on the adapter and the headset into the other jack.
2. Insert the enclosed prenatal CD of classical music in your CD player.
3. Position the Fetal Speakers against your belly. You will want to adjust the position of the speakers depending on what month of pregnancy you are in. You should be able to tell where your baby's head is by

where you feel her kicking most frequently. You may tuck the speakers into your clothing or use the Maternity Belt to hold them in place.

4. Play the music for approximately 10 minutes at a time and alternate the selections you play. Use the schedule included in this booklet as a record of what selections you have played and how frequently you played them.
5. Play the music just a little louder than you would normally play it for yourself. If it is much too loud for you, it will be too loud for your baby.

The Maternity Belt will provide you with support and additional comfort while you are doing something special for your baby. You may wear it when you are not playing music to your baby as well. Please care for it by hand washing it in cold water using a mild soap.

fig.12

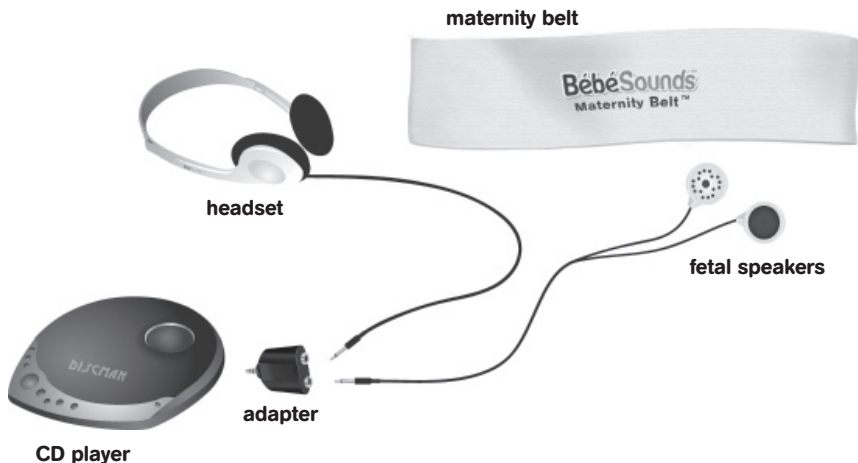PLAYING MUSIC TO YOUR NEWBORN BABY

The stimulation of your baby can continue after birth to help enhance development. Our postnatal CD contains special selections of music from a variety of classical composers. They are longer in duration than the prenatal CD because as your baby gets older she will be able to focus for longer periods of time.

INSTRUCTIONS FOR TALKING, SINGING, & READING TO YOUR UNBORN BABY

(See Fig. 13)

1. Connect the cable from the Fetal Speakers to the Fetal Microphone. The Fetal Microphone requires one AAA battery (not included).
2. Position the Fetal Speakers against your belly. You will want to adjust the position of the speakers depending on what month of pregnancy you are in. You should be able to tell where your baby's head is by where you feel her kicking most frequently. You may tuck the speakers into your clothing or use the Maternity Belt to hold them in place.
3. Be careful to speak in a normal voice. The microphone will amplify your voice so you don't have to speak too loudly as this may disturb your baby. You may also want to gently stroke your stomach while you are speaking.
4. Alternate your voice with the baby's father's voice – five minutes of yours and the five minutes of his. Speak in loving tones and repeat what you say 3 to 4 times. Repeat this a few times each day. Also read your baby a story. Choose one that you will read after she is born.

fig.13

FREQUENTLY ASKED QUESTIONS

Q: What type of music should I play?

A: The universal opinion seems to be that classical music is the best, particularly music by Mozart. That is why we have enclosed a 60-minute CD of specially selected music by Mozart.

Q: Do I have to use the Maternity Belt when playing the music?

A: Yes. For maximum benefit, the Fetal Speakers must be kept in a stationary position toward the bottom of your stomach as your baby has his/her head down in the third trimester. Our Maternity Belt is specially designed to accomplish this.

Q: How loud should I play the music or speak?

A: Because the sound must pass through the womb and be heard over the natural sounds of your body, play the music a little louder than you would for yourself. Speak in a normal voice as the microphone will amplify it.

Q: Are you sure I won't be disturbing my baby when I play music or speak to her?

A: Yes. What you will be doing is stimulating your baby and helping her brain develop. However, if you play the music or speak too long or too frequently, you will disturb your baby. Please remember your baby must have time to rest as well as develop.

Q: How soon can I start talking and playing music to my baby?

A: By the end of your fifth month of pregnancy, your baby's senses are developed enough so she can hear. Continue stimulating your baby from the end of the fifth month until she is born.

ENDNOTES

1. Jay Gordon, M.D. and Brenda Adderly, M.H.A., *Brighter Baby* (Washington, DC: Lifeline Press, 1999), 79.
2. Dr. Susan Ludington-Hoe with Susan K. Golant, *How to Have a Smarter Baby* (New York: Bantam Books, 1987), 39.
3. Dr. Susan Ludington-Hoe with Susan K. Golant, *How to Have a Smarter Baby* (New York: Bantam Books, 1987), 18.
4. Dr. Susan Ludington-Hoe with Susan K. Golant, *How to Have a Smarter Baby* (New York: Bantam Books, 1987), 18.
5. Dr. Sarah Brewer, *Super Baby* (London: Thorsons, 1998), 70.
6. Dr. Sarah Brewer, *Super Baby* (London: Thorsons, 1998), 97-98.
7. Dr. Sarah Brewer, *Super Baby* (London: Thorsons, 1998), 229.

SCHEDULE OF MUSIC PLAYED

date

time played

music played

SECTION III: ADDITIONAL PRODUCT INFORMATION

TIPS FOR USING THE PRENATAL HEART LISTENER®

- The Prenatal Heart Listener® has powerful amplification capabilities required to pick up the quiet sounds produced by your baby. To protect your ears and prevent feedback (squealing sounds), do not turn on the unit unless the listening cone is pressed against your body.
- There are many different sounds being produced by your baby and your body. Increasing the volume also increases these sounds. Keeping the volume at a moderate level usually produces the best results; therefore, adjust the volume carefully.
- Your baby's heartbeat is a very low volume sound and is very difficult to detect until the third trimester. Patience and very careful listening are required to locate the best position to hear the heartbeat. It is perfectly normal to hear it on some days and not others due to your baby's position and movements. It is also normal that you will hear it one moment and not the next. Don't worry! It is very natural for your baby to shift her position.

CARE & CLEANING

- To clean your Prenatal Heart Listener®, simply wipe with a soft dry or slightly damp cloth. Do not submerge in water or any other liquid.
- Store in a cool dry place.
- Keep out of the reach of children when not in use.
- Do not disassemble the Prenatal Heart Listener®.
- Only use a 9-volt alkaline battery.

- During long periods of non-use, remove battery from the compartment.
- Please dispose of batteries properly and safely.
- Do not use or place open flames such as lighted candles on or near the unit.
- The Prenatal Heart Listener® should be used in normal room temperature. Do not use in extreme heat or cold.

TROUBLESHOOTING

1. You cannot hear your unborn baby with your Prenatal Heart Listener®.
 - a. It may be too early in your pregnancy to hear your baby. Wait until you are in your third trimester when the fetal heartbeat starts becoming louder.
 - b. It is quite normal not to hear your baby due to her position. Try again later. The best time to listen is 3 to 4 hours after a meal and when you are relaxed. You may also want to try placing the Prenatal Heart Listener® against your lower back as this may help minimize interference from your own body's sounds.
 - c. Unit is not on. Turn unit on by turning the Power/Volume control wheel to the right until you hear a click and the red power indicator lights up.
 - d. Volume control is turned down. Increase the volume by turning the Power/Volume control wheel to the right.
 - e. The headset is not connected to the monitor. Make sure the headset is securely plugged into either jack on the monitor.
 - f. Battery is low. Red power indicator light will dim. Insert a new 9-volt alkaline battery.
 - g. Battery is not inserted properly. Open the battery compartment and check to see that positive (+) is matched with positive (+) and negative (-) is matched with negative (-).

- h. Foam cover is not in place on the listening cone. Make sure the foam cover is on the listening cone.
 - i. There is foreign matter in the listening cone. Remove the foam cover and make sure there is nothing in the listening cone.
2. There is feedback (squealing noises) when you use your Prenatal Heart Listener®.
- a. The volume is too high. Turn the volume down by turning the wheel to the left.
 - b. Lower the volume before moving the Prenatal Listener® against your skin.
3. You cannot record from your Prenatal Heart Listener®.
- a. The recording cable is not plugged in correctly. Make sure the L-shaped end of the recording cable is securely plugged into either jack on the Prenatal Listener® and the straight end into the line input of your recorder.
 - b. The recorder is not working. Make sure the recorder works. Replace batteries if necessary.
 - c. The volume control is turned down on your recorder. Increase the volume.
4. There is too much background noise and it makes a scratching sound when you move it.
- a. Turn the volume down BEFORE you move the Prenatal Heart Listener® from one spot to the next and don't turn it up again until you are no longer moving it against your stomach.

TECHNICAL SPECIFICATIONS

- For home use only.
- The Prenatal Heart Listener® requires one 9-volt battery (not included).
- The Fetal Microphone requires one AAA battery (not included).
- Manufacturer and battery information is also located on the inside wall of the battery compartment.

Applied Standards and Approvals

This product conforms to the provisions of the Electromagnetic Compatibility Directive (89/336/EEC).

The following standards apply to the design and/or manufacturing of this product:

- EN60065:2002: Audio, video and similar electronic apparatus. Safety requirements.
- EN55013:2001: Sound and television broadcast receivers and associated equipment. Radio disturbance characteristics. Limits and methods of measurement.
- EN55020:2002: Sound and television broadcast receivers and associated equipment. Immunity characteristics. Limits and methods of measurement.

DISCLAIMER

THE BÉBÉSOUNDS® PRENATAL HEART LISTENER® IS NOT A MEDICAL DEVICE.

The BébéSounds® Prenatal Heart Listener® is not a medical device and should not be used or relied upon in any way for diagnostic or other medical purposes. Any concerns about your pregnancy should be addressed with a medical practitioner. The BébéSounds® Prenatal Heart Listener® is not intended to be and should not be used as a substitute for regular prenatal care by an appropriate medical practitioner. You are encouraged to seek such prenatal care in connection with your pregnancy. The BébéSounds® Prenatal Heart Listener® should be used only in accordance with the instructions provided herein, and the manufacturers and distributors of the BébéSounds® Prenatal Heart Listener® are not responsible for inappropriate use. The manufacturer expressly disclaims liability for incidental, consequential, or special damages of any nature.

LIMITED PRODUCT WARRANTY

Register your product warranty online at www.bebesounds.com.

Unisar Inc., the distributor of the BébéSounds® Prenatal Heart Listener®, warrants to the original owner of this product that this product is free from defects in materials and workmanship for a period of three (3) months from the date of the original purchase. If this product fails to function properly when used as directed under normal conditions within the three-month period, Unisar Inc. will either repair or replace the product, at its option, free of charge, when it is returned in its original packaging, postage prepaid, with proof of purchase. Unisar Inc. will bear the cost of repairing or replacing the product and shipping it back to you within the United States. Repair or replacement of the defective item is subject to verification of the malfunction or defect when delivered to:

Unisar® Service Center
15 West 36th Street
New York, NY 10018

NOTE: The warrantor expressly disclaims liability for incidental, consequential, or special damages of any nature. This warranty does not cover 1) A defect that has been caused by repair or modification by any person other than a Unisar® technician; 2) A defect caused by negligence, accident, or misuse of the product; 3) A defect caused by incorrect installation of and/or defective batteries; 4) Battery life; or 5) Damage resulting from improper operation.

LEGAL RIGHTS VARY FROM STATE TO STATE.

NOTE: This warranty is only valid for products sold in the United States. If you have purchased your product outside of the United States, please see your local retailer or distributor for service.

If you believe your Prenatal Heart Listener® is defective or otherwise not operating properly, please check the battery first. Defective or exhausted batteries are the leading cause of problems with this item and are not covered by this limited warranty. If it is still not operating properly or if you have other technical problems, please contact the Bébésounds® Help Desk at 1-888-232-6476 Mon-Fri from 9:00AM-5:00PM EST.

The Bébésounds® Prenatal Heart Listener® is subject to United States patent and related foreign counterparts.

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