**NasalClear®**

Battery Operated Nasal Aspirator

**NasalClear®**

Aspirador nasal de pilas

**GRACO®**

## **OWNER'S MANUAL**

KEEP INSTRUCTIONS FOR FUTURE REFERENCE

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# **WARNING**

Failure to follow these warnings and the assembly instructions could result in serious injury or death.

- This product not to be used as a toy.
  - **WARNING:** This product is not a toy.
  - **Keep Nasal Aspirator away from heat sources**, such as fireplaces, radiators, stoves, and TVs or other appliances. Heat can damage the case or electrical parts.
  - **DO NOT** use the **Nasal Aspirator** Unit if:
    - Unit has been dropped or is damaged.
    - **DO NOT OPEN THE NASAL ASPIRATOR** No user-serviceable parts inside. Risk of electrical shock, fire or death.
    - Do not drop the instrument! Protect it from severe impact and shock.
- Keep in provided storage bag and store in a cool dry place.
- Keep out of the reach of children when not in use.
- NOTE: Do not microwave.
- CAUTION: Never insert the aspirator too far into the nostril. Never place the tip firmly against the inside wall of the nostril. Never use on ears, mouth, or eyes. This aspirator is for clearing congestion in nasal passages. Do not use this device for anything other than its intended purpose.
- NOTE: Please consult with your physician on when and how often you should aspirate your baby's nasal passages.
- Do not allow children to play with it. Individual parts may present a choking hazard.

## **FOR SAFE BATTERY USE:**

Keep the batteries out of children's reach.

Any battery may leak battery acid if mixed with a different battery type, if inserted incorrectly (put in backwards) or if all batteries are not replaced or recharged at the same time. Do not mix old and new batteries. Do not mix alkaline, standard (carbon-zinc) or rechargeable (nickel-cadmium) batteries.

Any battery may leak battery acid or explode if disposed of in fire or an attempt is made to charge a battery not intended to be recharged. Never recharge a cell of one type in a recharger made for a cell of another type.

Discard leaky batteries immediately. Leaky batteries can cause skin burns or other personal injury. When discarding batteries, be sure to dispose of them in the proper manner, according to your state and local regulations.

Remove batteries before storing product for a prolonged period of time. Batteries left in the unit may leak and cause damage.

Always remove batteries if the product is not going to be used for a month or more. Batteries left in the unit may leak and cause damage. Recommended type batteries-alkaline disposables, size AA (LR06). NEVER mix battery types. Change batteries when product ceases to operate satisfactorily.

- Batteries should be disposed of properly by recycling.

## **⚠ WARNING**

Modifications not authorized by the manufacturer may void users authority to operate this device.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception,

which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

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
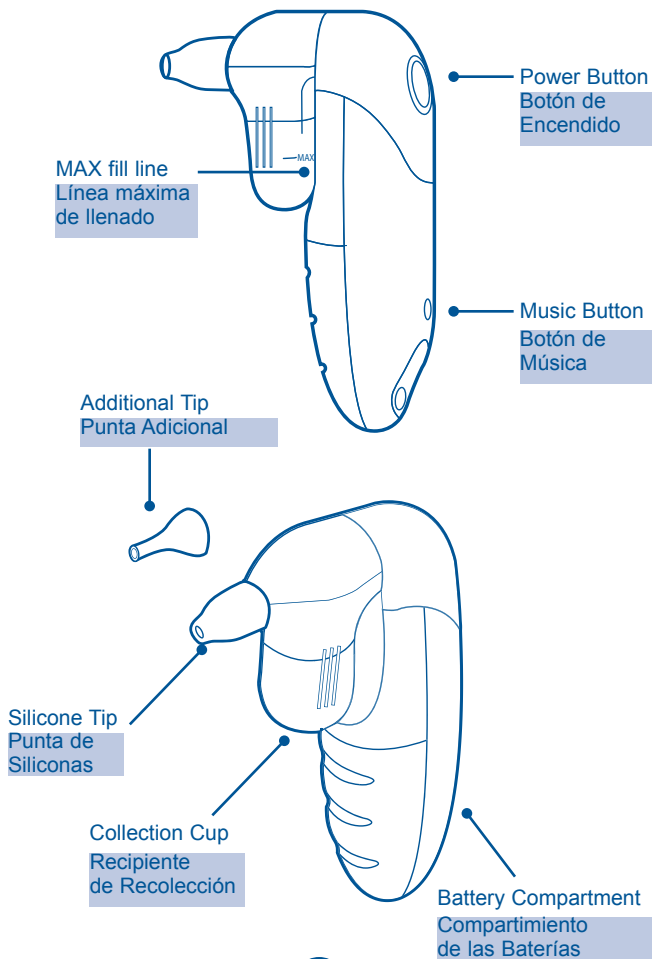
## Reseña

# Battery Installation

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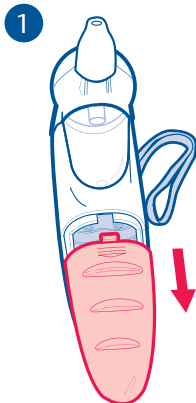
Remove the battery compartment cover as shown.

(Requires 2 AA (LR06) alkaline batteries, not included)

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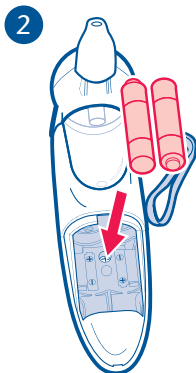Insert two (2) AA (LR06) alkaline batteries matching positive (+) to positive (+) and negative (-) to negative (-).

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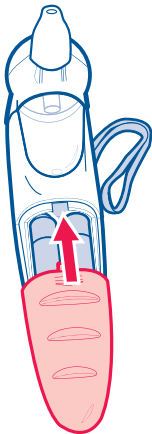
4

Replace the battery compartment cover as shown.

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
Replace batteries when the power or sound of the pump begins to sound weak during use.

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# Instructions for Use

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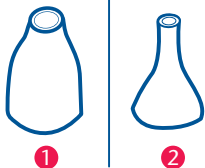1


Choose the correct tip to use based on the size and shape of your baby's nostrils. Also consider the type of congestion your baby has. The short tip **1** is recommended for use when the mucus is in a more solid form. The long, narrow tip **2** is recommended for use when the mucus is loose and in a liquid form.

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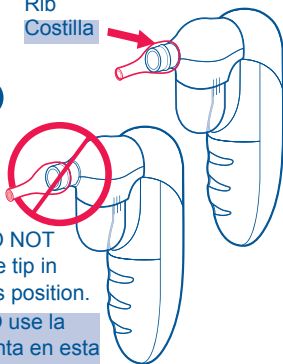2


DO NOT  
use tip in  
this position.

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Be sure to place the tip securely on the aspirator. Make sure tip is completely over the rib before using.

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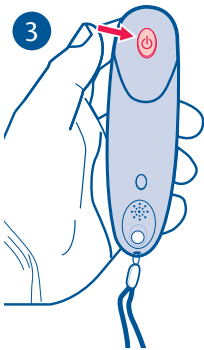
To activate, press and hold the power button to allow the suction to clear the mucus.

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Hold the baby upright and insert the tip into the baby's nose. **Be careful not to place the tip too far into the nostril or press against the inside wall of the nostril.** Be sure to keep the tip of the aspirator at a 90° angle. Do not point it up into the nostril. Allow the suction to clear the mucus. Repeat in other nostril and as needed.

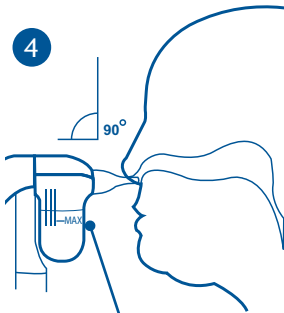**WARNING:** DO NOT allow mucus and or fluid to rise above the max fill line or halfway up height of collection cup.

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
**NOTE:** The mucus may not immediately appear in the collection cup. After each use it is recommended to do a quick clean of the suction head by removing the suction head from the unit.

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## Using Music Feature

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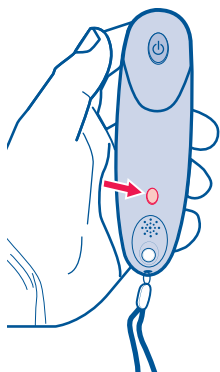If desired, press the MUSIC button to play a tune to comfort your baby. The music will automatically turn off after the tune is finished playing.

Press the MUSIC button again to play an additional tune.

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
# Care and Maintenance

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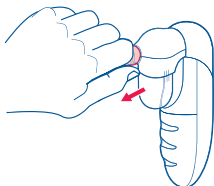
1


Separate the suction portion of the aspirator from the base by placing thumb under tip and peeling it off the aspirator.

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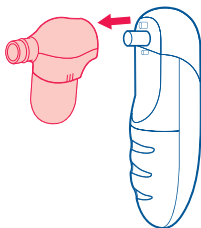2


3

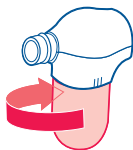Unscrew collection cup by turning it clockwise.

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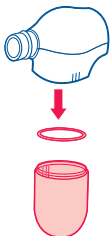Remove gray rubber ring located where collection cup screws in.

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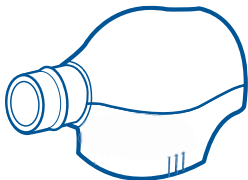
- Wash all 4 parts in warm soapy water or in the dishwasher (top shelf only) after each use.
- Dry all parts thoroughly before using again or assembling.
- Clean the base with a soft dry or slightly damp cloth.

**NOTE:** The base of the aspirator should never be submerged in water or any other liquid. Never put it in the dishwasher.

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
Rubber Ring  
Anillo de goma

- Be sure all parts fit tightly into each other before using.
- Remember to put the gray rubber ring back where the collection cup screws in. When screwing the collection cup back in, line up the three guides on each side of the collection cup with the three guides on the suction portion of the aspirator to ensure a proper fit.

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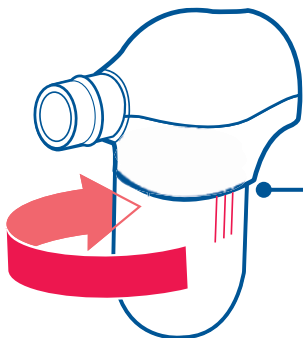Guides  
Guías

- Keep in provided storage bag and store in a cool dry place.
- Keep out of the reach of children when not in use.

**NOTE:** Do not microwave.

**CAUTION:** Never insert the aspirator too far into the nostril. Never place the tip firmly against the inside wall of the nostril. This aspirator is for clearing congestion in nasal passages. Do not use this device for anything other than its intended purpose. Do not modify this device in any way.

**NEVER use on ears, mouth, or eyes.**

**NOTE:** Please consult with your physician on when and how often you should aspirate your baby's nasal passages.

**WARNING:** This product is not a toy. Do not allow children to play with it. Individual parts may present a choking hazard.

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<b>Problem</b>	<b>Possible Cause</b>	<b>Solution</b>
The suction is weak or there is no suction at all.	Batteries are low or improperly installed.	Check or replace batteries.
	The gray rubber ring located where the collection cup screws in is missing.	Make sure there is a gray rubber ring where the collection cup screws in. Refer to page 14.
	The collection cup is not properly screwed in.	Make sure the three guides on each side of the collection cup line up with the guides on the suction portion of the aspirator. Refer to page 16.
The music will not play when the MUSIC button is pressed.	The tune is not finished.	The music will stop automatically after the tune is finished. Pressing the MUSIC button will not stop the music.

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**To purchase parts or accessories or for warranty information in the United States**, please contact us at the following:

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Graco Children's Products  
150 Oaklands Blvd.  
Exton, PA 19341

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To register your Graco product from within the U.S.A. visit us online at [www.gracobaby.com/productregistration](http://www.gracobaby.com/productregistration). We currently do not accept product registrations from those living outside the United States of America.

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©2010 Graco ISPD035BE 11/10